


3! | POB-MA


FLEX | HYDROPHOBIC PRELOADED

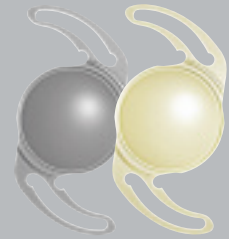

EASY, SAFE AND
PREDICTABLE
INJECTION SYSTEM



1st STEP Fill visco in the hole until the optic of the lens is covered (preferably HPMC)

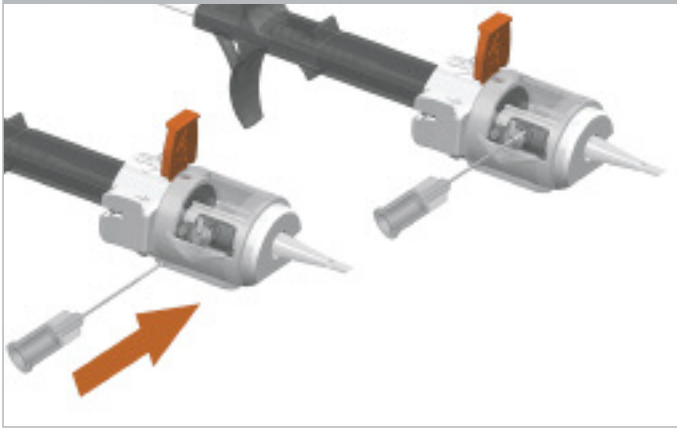


2nd STEP Rotate the transparent ring to close the cartridge



3rd STEP Pull out the red stopper



4th STEP Remove the ring-adaptor from the injector

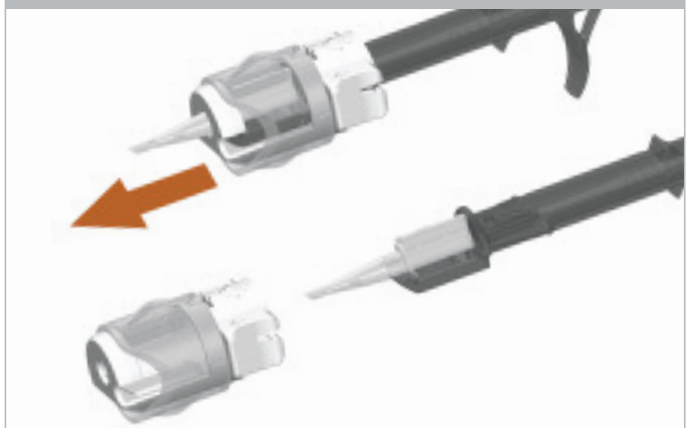

INJECTION Push the plunger slowly to the end position



Type	Single-piece Aspheric Hydrophobic Acrylic IOL
Material	Hydrophobic Acrylic with UV absorber / + optional blue light filter
Diameters	6.0 mm (optic) / 13.0 mm (overall)
Angulation	0° – asymmetrical design
Power range	-10.0D → +9.0D · (increment: 1.0 D) +10.0D → +30.0D · (increment: 0.5 D) +31.0D → +35.0D · (increment: 1.0 D)
A-Constant	118.9 (SRK/T)
Incision size	2.2 mm

* Patent pending

LA – IMCPOBMAENG 04 201608